

**SIMPLY CLEVER** SINGLE HAND USE



Montavit

# Cathejell Mono

## Specially designed for patients

- » Who are anaesthetised, e.g. general or local anaesthesia
- » Who are very sensitive, e.g. allergies
- » Where active substances are not desired, e.g. in vitro fertilisation
- » Where an anaesthetic effect is not tolerated, e.g. cardiovascular issues, taking beta-blockers



Single hand use  
Soft instillation

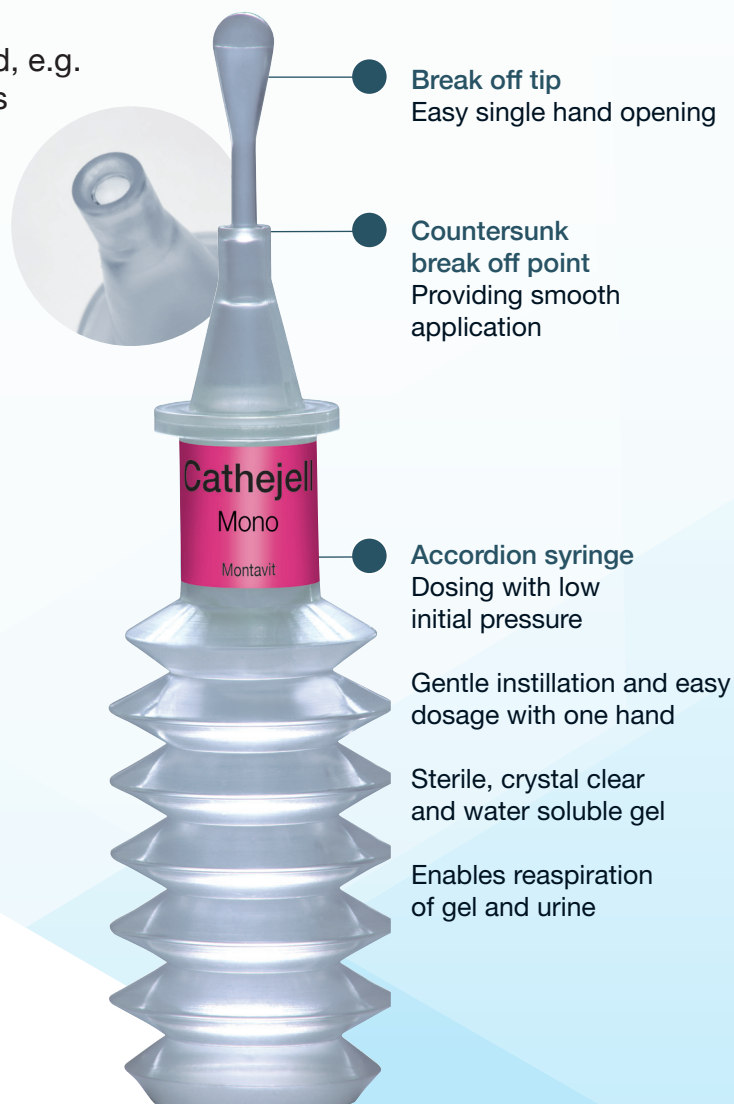
## Indications

Cathejell Mono is a sterile, water-soluble, clear lubricant gel for

- » Catheters
- » Endoscopes
- » Probes
- » Tubes
- » Tracheal intubation
- » Other medical instruments

## Key benefits

- » Gently unfolds the urethra
- » Excellent and long-lasting lubrication properties
- » Protects against injuries
- » High viscosity, no flow back from urethra\*
- » Free of parabens, PVC, latex and preservatives



\* Montavit, internal data on file

CE 0124

Cathejell Mono is a medical device class IIa and complies with the Essential Requirements of Directive 93/42/EEC and subsequent amendments.  
601112/042021



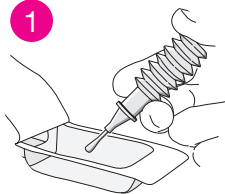
www.montavit.com

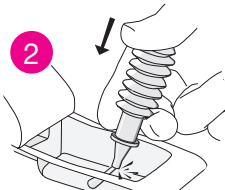
APPLICATION OF

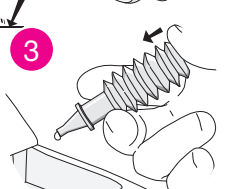
# Cathejell Mono

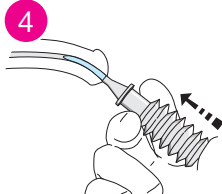
- » Forms a lubricating film between the mucosa and instrument
- » Protects the mucosa against injuries
- » Reduces (painful) friction

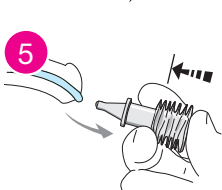
## Instillation into the urethra

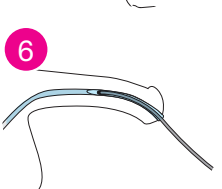
- 

1 Open blister pack by peeling back the paper
- 

2 Break off the snap-off tip onto sterile field
- 

3 Squeeze out one drop of gel with gentle pressure
- 

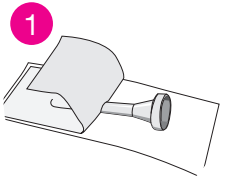
4 Insert the applicator nozzle into the meatus and instill slowly
- 

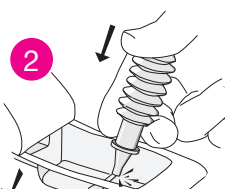
5 Empty the accordion syringe completely and remove it pressed
- 

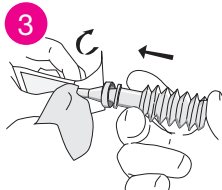
6 Introduce the catheter

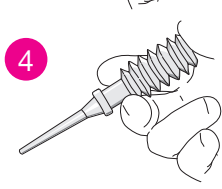
## Attach the applicator tip

*Cathejell's syringe tip can be extended by adding a sterile applicator tip, (Cathejell Cannula), for use in gynaecology or proctology.*

- 

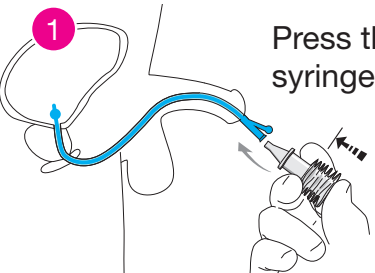
1 Open blister by peeling back the paper
- 

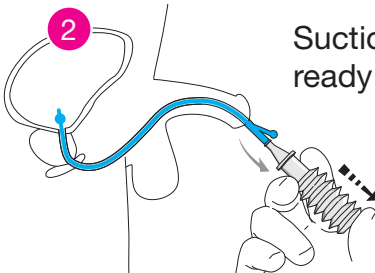
2 Break off the snap-off tip onto sterile field
- 

3 Attach the Cannula onto the accordion syringe
- 

4 Ready to use

## Reaspiration of gel and urine

- 

1 Press the accordion syringe
- 

2 Suction effect arises, ready for reaspiration